

Day Shift

OXYGEN AND SUCTION CHECKLIST FOR WALL MOUNTED AND PORTABLE UNITS

Please ensure checks are undertaken daily and date and sign below.

OXYGEN- mask and tubing assembled and attached to flow meter portable cylinders x4 are full and ready to use. Empty cylinders must be replaced. Contact porters.

SUCTION- ensure disposable hydrophobic filter is dated. Change filter post-use/if there is a colour change/annually. Ensure clean liner in VacSax canister. Switch suction on and kink tubing to ensure a rise in pressure.

Date Bed							
1							
2							
3							
4							
5							
6							
7							
8							
9							
10							
11							
12							
Bedside Gel							
Corridor Suction							
Portable o2 Cylinder							
Portable Suction							

**Equipment and checks
required at bedside:**

- 100% O2 mask
- Venturi 24% piece (blue) with O2 mask
- Nasal cannula
- Green tubing
- Suction tubing
- Yankauer sucker
- Check O2
- Check suction
- Hand gel
- Portable suction
- Portable O2

Date:							
1							
2							
3							
4							
5							
6							
7							
8							
9							
10							
11							
12							
13							
14							
15							
16							